ENGLISH | Instructions for Use

Disinfecting, Cleaning and Storing System

Contact lens disinfection system which disinfects, neutralizes, and removes protein deposits for all soft contact lenses including silicone hydrogel lenses.

Read this instruction sheet carefully before using **cleadew**, and keep it for future reference. To avoid eye infections or damage, always follow the user instructions.

COMPOSITION AND DESCRIPTION

Do not use **cleadew** if allergic to any of the ingredients used in **cleadew**.

cleadew 1 Lens case	cleadew 2 (Disinfecting, Neutralizing, and Cleaning Tablet)	cleadew 3 (Dissolving & Rinsing Solution)
Cap Filling Line 8 mL Lens holder R = Right L = Left	Symbol 1 is printed Disinfection Cleaning, Neutralization Symbol 2 is printed	Symbol 3 is printed
Povidone-iodine in the tablet quickly destroys harmful microorganisms, including Bacteria, Fungi, and Acanthamoeba. Next, the Ascorbic acid in the tablet neutralizes the povidone-iodine to a safe level. Finally, the included proteolytic enzyme complex removes protein deposits.	Povidone iodine (4.0 mg/tablet), Ascorbic acid (2.0 mg/tablet), and Proteolytic enzyme (0.5 mg/tablet)	This solution is used for dissolving cleadew 2 and rinsing lenses. Boric acid and Hydrogen peroxide as preservative

USER INSTRUCTIONS

-
-
-
-
-
-

Ensure that solution and lens are completely clear. If orange colour remains even after 4 hours, see **PRECAUTIONS** below.

STORAGE

After completing the above steps, in case of storing the lens for more than 24 hours, you should replace the solution with **cleadew 3**. You may store the lens in the unopened lens case up to 7 days, and then repeat the care steps from the beginning before wearing your lens.

WARNINGS

- Do not use **cleadew** if allergic to any of the ingredients used in **cleadew**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use **cleadew** with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using **cleadew** and keep it for future reference. To avoid eye infections, always follow the user instructions.
- Always wash and dry your hands completely before handling your lenses.
- If the solution does not turn orange after you have added a tablet, discard the solution, wash and dry the lens case, and then repeat the care steps from the beginning, in accordance with the user instructions.
- Do not wear your lenses if the orange stain persists.
- Consult your eye care practitioner if you have worn un-neutralized lenses.
- Ensure that solution and lens are completely clear before wearing the lenses. If orange color remains after 4 hours, lightly shake the lens case and leave it aside for 20-30 minutes. Check if the color has disappeared. If orange colour persists, discard solution and repeat care steps from the beginning.
- After each use, discard any **cleadew** solution that remains in the lens case.
- After each use of **cleadew**, wash the lens case with sterile saline solution and allow to air dry.
- Children should not use **cleadew** by themselves; parents are requested to be responsible for their daily care.
- Use only the specially designed lens case provided with **cleadew**.
- Discard any remaining solution 3 months after opening the bottle.
- Always use **cleadew 2** and **cleadew 3** together.
- Close bottle tightly after use.
- Do not use **cleadew** with hard or rigid gas permeable lenses.
- Do not swallow the solution or the tablets.
- Do not touch the tip of **cleadew 2** to avoid contamination.
- Do not use **cleadew** for heat disinfection.
- Do not use **cleadew 2** with any other container.
- Do not use **cleadew** after the expiry date.
- Do not use the solution or tablet if it is discoloured.
- Do not use **cleadew** if the tamper seal is damaged or missing.
- Do not use tap water to store and rinse the lenses.
- Keep out of reach of children.
- Secure the cap of **cleadew 3** after each use.
- Store products in a dry place at room temperature, avoiding direct sunlight.
- Do not replace **cleadew 2** with any other container.
- Do not use **cleadew 2** in its original package when not in use.
- Always use the new lens case provided.
- Use the tablet immediately after opening the package.
- Always store the tablets in their aluminum package when not in use.
- Do not change the contact lens care system without consulting eye care professional.

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