Ophtecs

ENGLISH | Instructions for Use

Disinfecting, cleaning and neutralizing system for all rigid gas permeable contact lenses.


Wash your hands well with mild soap before handling your contact lenses.

1 DISINFECTING, CLEANING AND NEUTRALIZING

- Place the lenses into the case's lens holders.
- Close the case lid on one side, put a neutralizing and cleaning tablet in, and fill up to 90% of the case with the orange disinfecting solution.
- Close the lid on the other side tightly.
- Soak the lenses for over four hours (or overnight).
- Confirm that the lenses and the solution are clear.

2 RINSING LENSES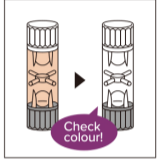

- Open the lid on one side and discard the solution in the lens case.
- Fill the case with sterile saline solution up to 60% of the case's capacity and close the lid tightly.
- Shake the case to rinse the lenses well.
- Discard the solution in the case and rinse the lenses again in the same way.

3 WEARING LENSES

- Remove the lenses from the holders and wear them.

4 WASHING LENS CASE

- After the care of the lenses is completed, wash the lens case with sterile saline solution and allow it to air dry.

STORING LENSES

- Store the lenses at room temperature after disinfecting, cleaning, and neutralizing with **cleadew GP**.
- Do not open the case until using the lenses.
- To store the lenses for 30days or longer, disinfect and clean with **cleadew GP** in accordance with Disinfecting, Cleaning and Neutralizing procedure as above every 30 days.
- Rinse the lenses with sterile saline solution before wearing them in accordance with rinsing lens procedure.

PRECAUTIONS

- ⚠️ To avoid serious eye infections or damage, always follow the user instructions.
- Use the disinfecting solution and the neutralizing and cleaning tablet together.
- Close the bottle tightly after use.
- Always use the **cleadew GP** system components including the lens case.
- Discard any remaining solution 90 days after opening the bottle.
- Do not use for soft contact lenses.
- Do not touch the tip of the bottle to avoid contamination.
- Do not swallow the solution or tablet. If you do so, drink a large amount of drinking water immediately.
- **⚠️ Not to be used in the eye.** If you do so, immediately rinse the eye with sterile saline solution or eye washes.
- Do not use the tablet if it is damaged or discoloured.
- Always use fresh solution. (Do not reuse solution.)
- Do not use the product after the expiry date.
- **⚠️ Do not use the product if the bottle or package is soiled or damaged.**
- Do not use the solution if it is cloudy or its colour is other colour than orange due to discolouration.
- Do not wear lenses without rinsing them with sterile saline solution after disinfecting, cleaning and neutralizing.
- Do not use the product together with another solution, tablet, or pharmaceutical drug.
- Do not use tap water to rinse the lenses or lens case.
- **⚠️** Keep out of reach of children.
- Do not transfer the solution to another bottle or case.
- To prevent contamination, use the new lens case in every purchase.
- Use the tablets immediately after opening the package.
- After using up all of the neutralizing and cleaning tablets, discard any remaining solution in the bottle.
- If the solution adheres to your clothes, wash with water immediately.
- Rinse off any remaining solution on the wash stand with water after each use to prevent staining on the surface of the wash stand.
- **⚠️** Store products in a dry place at room temperature, avoiding direct sunlight.
- If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use the product if you are allergic to any of the ingredients used in it.
- Do not change the contact lens care system without consulting your eye care professional.

INGREDIENTS

[Disinfecting solution] : [Neutralizing and cleaning tablet]
Povidone-iodine (0.05%) Sodium sulfite (2.4 mg/tablet) Proteolytic enzyme (8.0 mg/tablet)
STERILE A ⚠️

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