ENGLISH | Instructions for Use

Disinfecting, cleaning and storing system for soft contact lenses. FOR ALL SOFT CONTACT LENSES INCLUDING SILICONE HYDROGEL LENSES

Read this instruction sheet carefully before using cleadew, and keep it for future reference. To avoid eye infections or damage, always follow the user instructions.

COMPOSITION AND DESCRIPTION

Do not use cleadew if allergic to any of the ingredients used in cleadew.

Diagram showing the components of the cleadew system: Lenses case, Disinfecting/Cleaning/Neutralizing Tablet, Dropper & Rinsing Solution, and Lens holder. Includes instructions on how to use each component and lists ingredients like Povidone-iodine, Ascorbic acid, and Hydrogen peroxide.

USER INSTRUCTIONS

Step-by-step visual instructions for using cleadew: 1. Open lens holders, 2. Place tablet in case, 3. Soak lenses for 4 hours, 4. Check color, 5. Remove lenses, 6. Rinse lenses in palm.

STORAGE

After completing the above steps, in case of storing the lens for more than 24 hours, you should replace the solution with cleadew.

WARNINGS

- Do not use cleadew if allergic to any of the ingredients used in cleadew.
If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eyes, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
Do not use cleadew with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using cleadew and keep it for future reference.
Do not use tap water to store and rinse the lenses.
Keep out of reach of children.
Store products in a dry place at room temperature, avoiding direct sunlight.
Do not replace cleadew with any other container.
Do not use cleadew with any old lens case.
Always use the new lens case provided.
Use the tablet immediately after opening the package.
Always store the tablets in their aluminum package when not in use.
Do not change the contact lens care system without consulting your eye care professional.
Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women.

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Diagram showing the components of the cleadew system in French: Etui à lentilles, Comprimé, Désinfectant/Nettoyage/Neutrisation, and Solution. Includes instructions and ingredients in French.

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Diagram showing the components of the cleadew system in German: Kontaktlinsenbehälter, Tablette, Desinfektion/Reinigung/Neutralisation, and Lösung. Includes instructions and ingredients in German.

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