ENGLISH | Instructions for Use

Disinfecting, cleaning and storing system for soft contact lenses.
FOR ALL SOFT CONTACT LENSES INCLUDING SILICONE HYDROGEL LENSES

Read this instruction sheet carefully before using cleadew, and keep it for future reference. To avoid eye infections or damage, always follow the user instructions.

COMPOSITION AND DESCRIPTION

Do not use cleadew if allergic to any of the ingredients used in cleadew.

Table with 3 columns: Lenses case, Disinfecting, Cleaning, and Storing (Tablet), and Dissolving & Rinsing Solution. Includes diagrams of lens case, tablet, and lens holder, and lists ingredients like Povidone-iodine, Ascorbic acid, and Hydrogen peroxide.

USER INSTRUCTIONS

6-step illustrated instructions: 1. Open lens holders, 2. Place tablet in lens case, 3. Soak lenses for 4 hours, 4. Check colour, 5. Remove lenses, 6. Rinse lenses in palm.

STORAGE

After completing the above steps, in case of storing the lens for more than 24 hours, you should replace the solution with cleadew.

WARNINGS

- Do not use cleadew if allergic to any of the ingredients used in cleadew.
If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eyes, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
Do not use cleadew with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using cleadew and keep it for future reference.
Do not use tap water to store and rinse the lenses.
Keep out of reach of children.
Store products in a dry place at room temperature, avoiding direct sunlight.
Do not replace cleadew with any other container.
Do not use cleadew with any lens case.
Always use the new lens case provided.
Use the tablet immediately after opening the package.
Always store the tablets in their aluminium package when not in use.
Do not change the contact lens care system without consulting eye care professional.
Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women.
Be careful and avoid direct use of the solution in the eye; do not swallow.
Always use cleadew with the correct proper use.
Sensitivity to chemicals differs from person to person.
For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.
Discard any remaining solution 3 months after opening the bottle.
Always use cleadew and cleadew together.
Close bottle tightly after use.
Do not use cleadew with hard or rigid gas permeable lens.
Do not swallow the solution or the tablets.
Do not touch the tip of cleadew to avoid contamination.
Do not use cleadew for heat disinfection.
Do not use cleadew if broken.
Do not use tablet after the expiry date.
Do not use the solution or tablet if it is discoloured.

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