Optecs

ENGLISH | Instructions for Use

Disinfecting, cleaning and storing system for rigid gas permeable contact lenses.


FOR ALL RIGID GAS PERMEABLE CONTACT LENSES

Rinse the lenses with **cleadew CareSolution** or other rinsing solutions (sterile saline solution, rinsing solutions for contact lenses, and multipurpose solutions [MPS] for soft contact lenses) when wearing the lenses after care as required by this lens care system. Avoid using solutions containing sodium chlorite (for example, LENS Plus [Ocupure] or REGARD).

Wash your hands well with mild soap before handling your contact lenses.


Use the special lens case included (do not use any other lens cases).

1 DISINFECTING, CLEANING AND NEUTRALIZING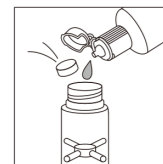

- Place the lenses into the case's lens holders.
- Close the case lid on one side, put a neutralizing and cleaning tablet in, and fill up to 90% of the case with the orange disinfecting solution.
- Failure to add the tablet may result in staining of the lenses. Be careful and do not forget to add the tablet to the lens case.
- Do not turn the lens case upside down after adding the tablet.
- Close the lid on the other side tightly.
- Soak the lenses for over four hours (or overnight).
- Allow the lenses to stand at room temperature since it may take time to neutralize the lenses at low temperatures.

2 RINSING LENSES

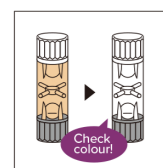- Confirm that the lenses and the solution are clear.
- Open the lid on one side and discard the solution in the lens case.
- Pick up the cleaned/disinfected lenses from the holder. Hold them with your fingers and rinse both sides with **cleadew CareSolution** or sterile saline solution for at least 10 seconds.
- If you fail to rinse the lenses or your rinsing was not thorough enough, the lenses may cause eye irritation. Be sure to rinse the lenses thoroughly.

3 WASHING LENS CASE

- After the care of the lenses is completed, wash the lens case with **cleadew CareSolution** or sterile saline solution and allow it to air dry.

STORING LENSES

- Store the lenses at room temperature after disinfecting, cleaning, and neutralizing with **cleadew GP**.
- Do not open the case until using the lenses.
- To store the lenses for 30 days or longer, disinfect and clean with **cleadew GP** in accordance with Disinfecting, Cleaning and Neutralizing procedure as above every 30 days.
- Rinse the lenses with **cleadew CareSolution** or sterile saline solution before wearing them in accordance with the rinsing lens procedure.

PRECAUTIONS

- Avoid serious eye infections or damage, always follow the user instructions.
- Do not use the product if you are allergic to any of the ingredients used in it.
- If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Always use the **cleadew GP** system components including the lens case.
- Use the disinfecting solution and the neutralizing and cleaning tablet together.
- Do not use the product together with another solution, tablet, or pharmaceutical drug.
- After using up all of the neutralizing and cleaning tablets, discard any remaining solution in the bottle.
- Do not use for soft contact lenses.
- Do not use the product if the bottle or package is soiled or damaged.
- Do not use the tablet if it is damaged or discoloured.
- Do not use the solution if it is cloudy or its colour is a colour other than orange due to discoloration.
- Do not wear lenses without rinsing them with **cleadew CareSolution** or sterile saline solution after disinfecting, cleaning and neutralizing.
- Do not reuse the solution. (The solution after lens care is neutralized and cannot be reused for the lens. To prevent eye infection, do not reuse the solution.)
- Do not use the product after the expiry date.
- Discard any remaining solution 90 days after opening the bottle.
- Use the tablets immediately after opening the package.
- Do not touch the tip of the bottle to avoid contamination.
- Close the bottle tightly after use.
- Do not transfer the solution to another bottle or case.
- Do not use tap water to rinse the lenses or lens case.
- Keep out of reach of children.
- Store products in a dry place at room temperature, avoiding direct sunlight.
- To prevent contamination, use a new lens case with each purchase.
- If the solution adheres to your clothes, wash with water immediately.
- Rinse off any remaining solution on the wash stand with water after each use to prevent staining on the surface of the wash stand.
- Do not swallow the solution or tablet. If you do so, drink a large amount of drinking water immediately.
- Do not be used in the eye. If you do so, immediately rinse the eye with sterile saline solution or eye washes.
- Do not change the contact lens care system without consulting your eye care professional.
- Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Although this product also contains boric acid, the amount of boric acid in this product is at a concentration that does not pose a health hazard. This product is safe for use by children and pregnant or lactating women as long as the product's directions are followed. Sensitivity to chemicals differs from person to person. For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.

INGREDIENTS

[Disinfecting solution]
Povidone-iodine (0.05%)
Boric acid

[Neutralizing and cleaning tablet]
Sodium sulfate (2.4 mg/tablet),
Proteolytic enzyme (8.0 mg/tablet)

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