
## ENGLISH | Instructions for Use

### Rinsing solution for contact lens.

FOR ALL RIGID GAS PERMEABLE CONTACT LENSES INCLUDING SCLERAL CONTACT LENSES

Wash your hands well with mild soap before handling your contact lenses.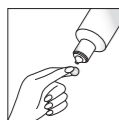

Pick up the cleaned / disinfected lenses from the holder. Hold them with your fingers and rinse both sides with the **cleadew CareSolution** for at least 10 seconds.

### WARNINGS

- Do not use **cleadew CareSolution** if allergic to any of the ingredients used in **cleadew CareSolution**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.

### PRECAUTIONS

- Read this instruction sheet carefully before using **cleadew CareSolution** and keep it for future reference. To avoid eye infections, always follow the user instructions.
- To prevent eye infection, do not reuse the solution.
- Children should not use **cleadew CareSolution** by themselves; parents are requested to be responsible for their daily care.
- Discard any remaining solution 3 months after opening the bottle.
- Close bottle tightly after use.
- Do not use for soft contact lenses.
- cleadew CareSolution** can not disinfect contact lenses.
- Do not use **cleadew CareSolution** for the storage of contact lenses.
- Do not swallow the solution. If you do so, drink a large amount of drinking water immediately.
- Do not touch the tip of **cleadew CareSolution** to avoid contamination.
- Do not use **cleadew CareSolution** after the expiry date.
- Do not use the solution if it is discoloured.
- Do not use **cleadew CareSolution** if the tamper seal is damaged or missing.
- Do not use tap water to store and rinse the lenses.
- Keep out of reach of children.
- Store products at room temperature, avoiding direct sunlight.
- To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent.
- Do not change the contact lens care system without consulting eye care professional.
- Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Be careful and avoid direct use of the solution in the eye; do not swallow. Proper use can reduce the risk; therefore, **cleadew CareSolution** can be used by children or pregnant or lactating women. We recommend rinsing the lenses with boric acid-free sterile physiological saline before wearing them if you are susceptible.

### INGREDIENTS

**cleadew CareSolution** Hydrogen peroxide as preservative (0,004%) Boric acid

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**OPHTECS Europe B.V.**