ENGLISH | Instructions for Use

Disinfecting, cleaning and storing system for soft contact lenses.

FOR ALL SOFT CONTACT LENSES INCLUDING SILICONE HYDROGEL LENSES

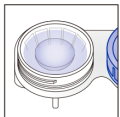
Wash your hands well with mild soap before handling your contact lenses.

1 CLEANING

- Remove each lens from your eye and place it on your palm. Place a few drops of **cleadew MPS** on each lens surface and rub 20-30 times with your finger. Rinse each lens thoroughly for 5 seconds with fresh **cleadew MPS**.

2 DISINFECTING

- Fill the lens case with **cleadew MPS** and place the lenses into the lens case. Close the lens case tightly. Soak lenses for at least 4 hours (or overnight) until ready to wear.

3 RINSING AND WEARING LENSES

- Rinse lenses for 5 seconds with **cleadew MPS** before wearing.

4 WASHING LENS CASE

- After the lens care, rinse the lens case with **cleadew MPS** and allow to air dry.

STORAGE

- Lenses may be stored in the unopened lens case for a maximum of 30 days. If the lenses are stored for longer, replace with fresh **cleadew MPS** every 30 days and before wearing your lenses.

WARNINGS

- Do not use **cleadew MPS** if allergic to any of the ingredients used in **cleadew MPS**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use **cleadew MPS** with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using **cleadew MPS** and keep it for future reference. To avoid eye infections, always follow the user instructions.
- Do not reuse the solution. (The solution remaining in the lens case after lens care cannot sufficiently disinfect the lens. To prevent eye infection, do not reuse the solution).
- After each use, discard any **cleadew MPS** solution that remains in the lens case.
- Children should not use **cleadew MPS** by themselves; parents are requested to be responsible for their daily care.
- Discard any remaining solution 3 months after opening the bottle.
- Close bottle tightly after use.
- Do not use **cleadew MPS** with hard or rigid gas permeable lenses.
- Do not swallow the solution.
- Do not touch the tip of **cleadew MPS** to avoid contamination.
- Do not use **cleadew MPS** for heat disinfection.
- Do not use **cleadew MPS** after the expiry date.
- Do not use the solution if it is discoloured.
- Do not use **cleadew MPS** if the tamper seal is damaged or missing.
- Do not use tap water to store and rinse the lenses.
- Keep out of reach of children.
- Store products at room temperature, avoiding direct sunlight.
- To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent.
- Do not use **cleadew MPS** with an old lens case.
- Always use the new lens case provided.
- Do not change the contact lens care system without consulting eye care professional.
- Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Be careful and avoid direct use of the solution in the eye; do not swallow. Proper use can reduce the risk; therefore, **cleadew MPS** can be used by children or pregnant or lactating women. Sensitivity to chemicals differs from person to person. For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.

INGREDIENTS

cleadew MPS
Polyhexamethylene biguanide hydrochloride (0,00011%)
Alexidine dihydrochloride (0,0004%)

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OPHTECS Europe B.V.
OPHTECS Europe B.V. Marithaime 20, 6662 WD Elst, The Netherlands