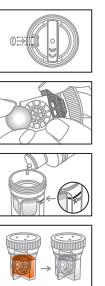
ENGLISH | Instructions for Use

Disinfecting, cleaning and neutralizing system for scleral contact lenses.

Wash your hands well with mild soap before handling your contact lenses.

1 DISINFECTING, CLEANING AND NEUTRALIZING

- Set a neutralizing and cleaning tablet at the specified position (slit) of the dedicated case body. Be sure to use the dedicated case. ▲ Setting the tablet at a position other than the specified position may lead to unsuccessful neutralisation, resulting in colouration of the lens.
- Place the lenses in the lens holder of the case.
- Fill the case with the disinfecting solution up to the marked line.
- Close the case lid and let it stand for at least four hours.
- Confirm that the lenses and the solution are clear.

2 RINSING LENSES

- Discard the solution after neutralisation, fill the case with sterile physiological saline up to the marked line, close the lid, and shake the case to rinse the lenses well. ▲ Do not use any solution (Lens Plus, etc.) containing oxidative preservative such as OcuPure®.
- Discard the solution in the case and rinse the lenses again in the same way.

3 WEARING LENSES

- Remove the lenses from the holders and wear them.

STORING LENSES

- To store the lenses, be sure to follow the instructions of eye care professionals.
- When planning to wear lenses that have been stored for 24 hours or longer, it is recommended to start care from the beginning before wearing them.

PRECAUTIONS

- Avoid serious eye infections or damage, always follow the user instructions.
- Use the disinfecting solution and the neutralizing and cleaning tablet together.
- Close the bottle tightly after use.
- Always use the cleadew SL system components including the lens case.
- Discard any remaining solution 90 days after opening the bottle.
- Do not use for soft contact lenses.
- Do not touch the tip of the bottle to avoid contamination.
- Do not swallow the solution or tablet. If you do so, drink a large amount of drinking water immediately.
- Not to be used in the eye. If you do so, immediately rinse the eye with sterile saline solution or eye washes.
- Do not use the tablet if it is damaged or discoloured.
- Always use fresh solution. (Do not reuse solution.)
- Do not use the product after the expiry date.
- Do not use the product if the bottle or package is soiled or damaged.
- Do not use the solution if it is cloudy or its colour is other colour than orange due to discolouration.
- Do not wear lenses without rinsing them with sterile saline solution after disinfecting, cleaning and neutralizing.
- Do not use the product together with another solution, tablet, or pharmaceutical drug.
- Do not use tap water to rinse the lenses or lens case.
- Keep out of reach of children.
- To prevent bacterial contamination, do not transfer the disinfecting solution to another container, nor replenish it with water or other liquid medical agent.
- To prevent contamination, use the new lens case in every purchase.
- Use the tablets immediately after opening the package.
- After using up all of the neutralizing and cleaning tablets, discard any remaining solution in the bottle.
- If the solution adheres to your clothes, wash with water immediately.
- Rins off any remaining solution on the wash stand with water after each use to prevent staining on the surface of the wash stand.
- Store products in a dry place at room temperature, avoiding direct sunlight.
- If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use the product if you are allergic to any of the ingredients used in it.
- Do not change the contact lens care system without consulting your eye care professional.
- In the unlikely event that the lens is coloured after care, do not wear the coloured lens.

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