# Ophtecs

ENGLISH | Instructions for Use

Disinfecting, cleaning and neutralizing system for all rigid gas permeable contact lenses.


Wash your hands well with mild soap before handling your contact lenses.

## 1 DISINFECTING, CLEANING AND NEUTRALIZING

- Place the lenses into the case's lens holders.
- Close the case lid on one side, put a neutralizing and cleaning tablet in, and fill up to 90% of the case with the orange disinfecting solution.
- Close the lid on the other side tightly.
- Soak the lenses for over four hours (or overnight).
- Confirm that the lenses and the solution are clear.

## 2 RINSING LENSES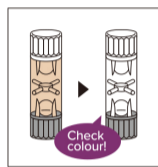

- Open the lid on one side and discard the solution in the lens case.
- Fill the case with unbuffered, non-preservative sterile saline solution up to 60% of the case's capacity and close the lid tightly.
- Gently shake the case to rinse the lenses well.
- Discard the solution in the case and rinse the lenses again in the same way.
  - ⚠ Avoid using saline or MPS solutions containing sodium chlorite as a preservative or disinfectant.

## 3 WEARING LENSES

- Remove the lenses from the holders and wear them.

## 4 WASHING LENS CASE

- After the care of the lenses is completed, wash the lens case with unbuffered, non-preservative sterile saline solution and allow it to air dry.

## STORING LENSES

- Store the lenses at room temperature after disinfecting, cleaning, and neutralizing with **cleadew GP**.
- Do not open the case until using the lenses.
- To store the lenses for 30days or longer, disinfect and clean with **cleadew GP** in accordance with Disinfecting, Cleaning and Neutralizing procedure as above every 30 days.
- Rinse the lenses with unbuffered, non-preservative sterile saline solution before wearing them in accordance with the rinsing lens procedure.

## ⚠ PRECAUTIONS

- ⚠ To avoid serious eye infections or damage, always follow the user instructions.
- ⚠ Use the disinfecting solution and the neutralizing and cleaning tablet together.
- ⚠ Close the bottle tightly after use.
- ⚠ Always use the **cleadew GP** system components including the lens case.
- ⚠ Discard any remaining solution 90 days after opening the bottle.
- ⚠ Do not use for soft contact lenses.
- ⚠ Do not touch the tip of the bottle to avoid contamination.
- ⚠ Do not swallow the solution or tablet. If you do so, drink a large amount of drinking water immediately.
- ⚠ Not to be used in the eye. If you do so, immediately rinse the eye with unbuffered, non-preservative sterile saline solution or eye washes.
- ⚠ Do not use the tablet if it is damaged or discoloured.
- ⚠ Always use fresh solution. (Do not reuse solution.)
- ⚠ Do not use the product after the expiry date.
- ⚠ Do not use the product if the bottle or package is soiled or damaged.
- ⚠ Do not use the solution if it is cloudy or its colour is a colour other than orange due to discoloration.
- ⚠ Do not wear lenses without rinsing them with unbuffered, non-preservative sterile saline solution after disinfecting, cleaning and neutralizing.
- ⚠ Do not use the product together with another solution, tablet, or pharmaceutical drug.
- ⚠ Do not use tap water to rinse the lenses or lens case.
- ⚠ Keep out of reach of children.
- ⚠ Do not transfer the solution to another bottle or case.
- ⚠ To prevent contamination, use a new lens case with each purchase.
- ⚠ Use the tablets immediately after opening the package.
- ⚠ After using up all of the neutralizing and cleaning tablets, discard any remaining solution in the bottle.
- ⚠ If the solution adheres to your clothes, wash with water immediately.
- ⚠ Rinse off any remaining solution on the wash stand with water after each use to prevent staining on the surface of the wash stand.
- ⚠ Store products in a dry place at room temperature, avoiding direct sunlight.
- ⚠ If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- ⚠ Do not use the product if you are allergic to any of the ingredients used in it.
- ⚠ Do not change the contact lens care system without consulting your eye care professional.

## INGREDIENTS

[Disinfecting solution] Povidone-iodine (0.05%)	[Neutralizing and cleaning tablet] Sodium sulfite (2.4 mg/tablet) Proteolytic enzyme (8.0 mg/tablet)
--	---

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OPHTECS Europe B.V.