
ENGLISH | Instructions for Use

Multi-Purpose Solution for disinfecting, cleaning, rinsing and storing for all soft contact lenses


Wash your hands well with mild soap before handling your contact lenses.

1 CLEANING

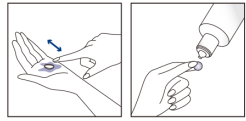- Remove each lens from your eye and place it on your palm. Place a few drops of **cleadew MPS** on each lens surface and rub 20-30 times with your finger. Rinse each lens thoroughly for 5 seconds with fresh **cleadew MPS**.

2 DISINFECTING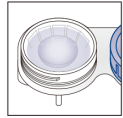

- Fill the lens case with **cleadew MPS** and place the lenses into the lens case. Close the lens case tightly. Soak lenses for at least 4 hours (or overnight) until ready to wear.

3 RINSING AND WEARING LENSES

- Rinse lenses for 5 seconds with **cleadew MPS** before wearing.

4 WASHING LENS CASE

- After the lens care, rinse the lens case with **cleadew MPS** and allow to air dry.

STORAGE

- Lenses may be stored in the unopened lens case for a maximum of 30 days. If the lenses are stored for longer, replace with fresh **cleadew MPS** every 30 days and before wearing your lenses.

WARNINGS

- Do not use **cleadew MPS** if allergic to any of the ingredients used in **cleadew MPS**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use **cleadew MPS** with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using **cleadew MPS** and keep it for future reference. To avoid eye infections, always follow the user instructions.
- Always wash and dry your hands completely before handling your lenses.
- After each use, discard any **cleadew MPS** solution that remains in the lens case.
- Children should not use **cleadew MPS** by themselves; parents are requested to be responsible for their daily care.
- Discard any remaining solution 3 months after opening the bottle.
- Close bottle tightly after use.
- Do not use **cleadew MPS** with hard or rigid gas permeable lenses.
- Do not swallow the solution.
- Do not touch the tip of **cleadew MPS** to avoid contamination.
- Do not use **cleadew MPS** for heat disinfection.
- Do not use **cleadew MPS** after the expiry date.
- Do not use the solution if it is discoloured.
- Do not use **cleadew MPS** if the tamper seal is damaged or missing.
- Do not use tap water to store and rinse the lenses.
- Keep out of reach of children.
- Secure the cap of **cleadew MPS** after each use.
- Store products at room temperature, avoiding direct sunlight.
- To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent.
- Do not use **cleadew MPS** with an old lens case.
- Always use the new lens case provided.
- Do not change the contact lens care system without consulting eye care professional.

INGREDIENTS

cleadew MPS
 Polyhexamethylene biguanide hydrochloride (0.00011%)
 Alexidine dihydrochloride (0.0004%)
 Poloxamer

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