ENGLISH | Instructions for Use

Rinsing solution for contact lens.

FOR ALL CONTACT LENSES

Wash your hands well with mild soap before handling your contact lenses.

Pick up the cleaned / disinfected lenses from the lens case. Hold them with your fingers and rinse both sides with the **cleadow CareSolution** for at least 10 seconds.

WARNINGS

- Do not use **cleadow CareSolution** if allergic to any of the ingredients used in **cleadow CareSolution**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.

PRECAUTIONS

- Read this instruction sheet carefully before using **cleadow CareSolution** and keep it for future reference. To avoid eye infections, always follow the user instructions.
- To prevent eye infection, do not reuse the solution.
- Children should not use **cleadow CareSolution** by themselves; parents are requested to be responsible for their daily care.
- Discard any remaining solution 3 months after opening the bottle.
- Close bottle tightly after use.
- cleadow CareSolution** can not disinfect contact lenses.
- Do not use **cleadow CareSolution** for the storage of contact lenses.
- Do not swallow the solution. If you do so, drink a large amount of drinking water immediately.
- Do not touch the tip of **cleadow CareSolution** to avoid contamination.
- Do not use **cleadow CareSolution** after the expiry date.
- Do not use the solution if it is discoloured.
- Do not use **cleadow CareSolution** if the tamper seal is damaged or missing.
- Do not use tap water to store and rinse the lenses.
- Keep out of reach of children.
- Store products at room temperature, avoiding direct sunlight.
- To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent.
- Do not change the contact lens care system without consulting eye care professional.
- Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Although this product also contains boric acid, the amount of boric acid in this product is at a concentration that does not pose a health hazard. This product is safe for use by children and pregnant or lactating women as long as the product's directions are followed. Sensitivity to chemicals differs from person to person. For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.

INGREDIENTS

cleadow CareSolution

Hydrogen peroxide as preservative (0,004%)
⚠Boric acid

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OPHTECS Europe B.V.