
ENGLISH | Instructions for Use

Disinfecting, cleaning and storing system for rigid gas permeable contact lenses.

FOR ALL RIGID GAS PERMEABLE CONTACT LENSES


Wash your hands well with mild soap before handling your contact lenses.

1 CLEANING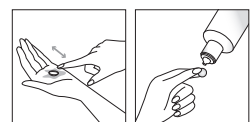

- Remove each lens from your eye and place it on your palm. Place a few drops of **cleadew GP hydra one** on each lens surface and rub 20-30 times with your finger. Rinse each lens thoroughly for 5 seconds with fresh **cleadew GP hydra one**.

2 DISINFECTING

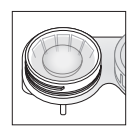- Fill the lens case with **cleadew GP hydra one** and place the lenses into the lens case. Close the lens case tightly. Soak lenses for at least 4 hours (or overnight) until ready to wear.

3 RINSING AND WEARING LENSES

- Rinse lenses for 5 seconds with **cleadew GP hydra one** before wearing.

4 WASHING LENS CASE

- After the lens care, rinse the lens case with **cleadew GP hydra one** and allow to air dry.

STORAGE

- Lenses may be stored in the unopened lens case for a maximum of 30 days. If the lenses are stored for longer, replace with fresh **cleadew GP hydra one** every 30 days and before wearing your lenses.

WARNINGS

- Do not use **cleadew GP hydra one** if allergic to any of the ingredients used in **cleadew GP hydra one**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use **cleadew GP hydra one** with any eye medication or other contact lens care products.

PRECAUTIONS

- 

Read this instruction sheet carefully before using **cleadew GP hydra one** and keep it for future reference. To avoid eye infections, always follow the user instructions.
- 

Do not reuse the solution. (The solution remaining in the lens case after lens care cannot sufficiently disinfect the lens. To prevent eye infection, do not reuse the solution).
- 

After each use, discard any **cleadew GP hydra one** solution that remains in the lens case.
- 

Children should not use **cleadew GP hydra one** by themselves; parents are requested to be responsible for their daily care.
- 

Discard any remaining solution 3 months after opening the bottle.
- 

Close bottle tightly after use.
- 

Do not use for soft contact lenses.
- 

Do not swallow the solution.
- 

Do not touch the tip of **cleadew GP hydra one** to avoid contamination.
- 

Do not use **cleadew GP hydra one** for heat disinfection.
- 

Do not use **cleadew GP hydra one** after the expiry date.
- 

Do not use the solution if it is discoloured.
- 

Do not use **cleadew GP hydra one** if the tamper seal is damaged or missing.
- 

Do not use tap water to store and rinse the lenses.
- 

Keep out of reach of children.
- 

Store products at room temperature, avoiding direct sunlight.
- 

To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent.
- 

Do not use **cleadew GP hydra one** with an old lens case.
- 

Always use the new lens case provided.
- 

Do not change the contact lens care system without consulting eye care professional.
- 

Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Although this product also contains boric acid, the amount of boric acid in this product is at a concentration that does not pose a health hazard. This product is safe for use by children and pregnant or lactating women as long as the product's directions are followed. Sensitivity to chemicals differs from person to person. For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.
- 

Do not use **cleadew GP hydra one** after the expiry date.
- 

Do not use **cleadew GP hydra one** if the tamper seal is damaged or missing.
- 

Do not use tap water to store and rinse the lenses.
- 

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OPHTECS Europe B.V.