
ENGLISH | Instructions for Use

Comfort solution for contact lens wear.
FOR SCLERAL CONTACT LENSES

Wash your hands with mild soap before handling your scleral contact lenses.
Disinfect and rinse the lens with the solution recommended by your eye care professional.

1 Open the unidose by twisting off the top, taking care to hold it in an upright position.

2 Fill the scleral contact lens with **cleadew SLi** before inserting in the eye.
*** Be careful not to leave air bubbles.**

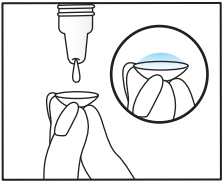3 Discard the unidose after use. Do not re-use.

WARNINGS

- Do not use the product if you are allergic to any of the ingredients used in it.
- If you experience such symptoms as eye pain and irritation, remove your lenses immediately and consult your eye doctor or an eye care practitioner.

PRECAUTIONS

- To avoid serious eye infections or damage, always follow your eye care professional's directions and the user instructions.
- cleadew SLi** is for single use only. Discard any remaining solution. (**cleadew SLi** is preservative-free. Any solution remaining in the case may be contaminated by microorganisms. To prevent eye infection, do not reuse the solution remaining in the container once opened).
- Do not use for soft contact lenses.
- cleadew SLi** can not disinfect contact lenses.
- Do not use **cleadew SLi** for the storage of contact lenses.
- Do not swallow the solution. If you do so, drink a large amount of drinking water immediately.
- Do not use the product after the expiry date.
- Do not use the product if the unidose is leaked or damaged.
- Do not use the solution if it is cloudy or discoloured.
- Keep out of reach of children.
- To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent.
- Store products at room temperature, avoiding direct sunlight.
- Do not change the contact lens care system without consulting your eye care professional.
- Visit your eye care professional regularly.

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OPHTECS Europe B.V.