Ophtecs

ENGLISH | Instructions for Use


PLEASE READ CAREFULLY AND KEEP THIS INFORMATION FOR FUTURE USE

Disinfecting, cleaning and storing system for rigid gas permeable contact lenses.

FOR ALL RIGID GAS PERMEABLE CONTACT LENSES


Wash your hands well with mild soap before handling your contact lenses.

CLEANING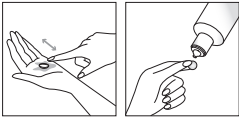

Remove each lens from your eye and place it on your palm. Place a few drops of **cleadow GP hydra one** on each lens surface and rub 20-30 times with your finger. Rinse each lens thoroughly for 5 seconds with fresh **cleadow GP hydra one**.

DISINFECTING

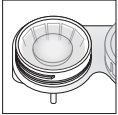Fill the lens case with **cleadow GP hydra one** and place the lenses into the lens case. Close the lens case tightly. Soak lenses for at least 4 hours (or overnight) until ready to wear.

RINSING AND WEARING CORNEAL LENSES

Rinse corneal GP lenses for 5 seconds with **cleadow GP hydra one** before wearing

RINSING AND WEARING SPECIALTY LENSES (SCLERAL LENSES OR ORTHOKERATOLOGY LENSES)

For specialty lens users (scleral lenses or orthokeratology/Ortho K lenses), it is required to rinse your lenses thoroughly for approximately 5 seconds using sterile saline or an appropriate lens filling solution before placing them on your eye. Do not use **cleadow GP hydra one** as an insertion or rinsing solution for scleral or Ortho K lenses. Always use sterile saline or another insertion solution specifically recommended by your eye care practitioner when inserting your lenses.

WASHING LENS CASE

After the lens care, rinse the lens case with **cleadow GP hydra one** and allow to air dry.

STORAGE

Lenses may be stored in the unopened lens case for a maximum of 30 days. If the lenses are stored for longer, replace with fresh **cleadow GP hydra one** every 30 days and before wearing your lenses.

WARNINGS

Do not use **cleadow GP hydra one** if allergic to any of the ingredients used in **cleadow GP hydra one**. If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner. Do not use **cleadow GP hydra one** with any eye medication or other contact lens care products. Do not use **cleadow GP hydra one** as an insertion or rinsing solution for **specialty lenses (scleral or orthokeratology lenses)**.

PRECAUTIONS

Read this instruction sheet carefully before using **cleadow GP hydra one** and keep it for future reference. To avoid eye infections, always follow the user instructions. Take special care to follow these instructions when using **cleadow GP hydra one** with **SPECIALTY LENSES (SCLERAL OR ORTHOKERATOLOGY LENSES)**. Do not reuse the solution; after each use, discard any **cleadow GP hydra one** solution remaining in the lens case. Children should not use **cleadow GP hydra one** by themselves; parents are requested to be responsible for their daily care. Discard any remaining solution 3 months after opening the bottle. Close bottle tightly after use. Do not use for soft contact lenses. Do not swallow the solution. Do not touch the tip of **cleadow GP hydra one** to avoid contamination. Do not use **cleadow GP hydra one** for heat disinfection. Do not use **cleadow GP hydra one** after the expiry date. Do not use the solution if it is discoloured. Do not use **cleadow GP hydra one** if the tamper seal is damaged or missing. Do not use tap water to store and rinse the lenses. Keep out of reach of children. Store products at room temperature, avoiding direct sunlight. To prevent bacterial contamination, do not transfer the solution to another container, nor replenish it with water or other liquid medical agent. Do not use **cleadow GP hydra one** with an old lens case. Always use the new lens case provided. Do not change the contact lens care system without consulting eye care practitioner. Boric acid is toxic to reproduction and can reduce future fertility if taken by children or harm babies if taken by pregnant or lactating women. Although this product also contains boric acid, the amount of boric acid in this product is at a concentration that does not pose a health hazard. This product is safe for use by children and pregnant or lactating women as long as the product's directions are followed. Sensitivity to chemicals differs from person to person. For sensitive individuals, the recommendation is to rinse the lens with sterile saline solution before wearing.

INGREDIENTS

cleadow GP hydra one
Polyhexamethylene biguanide hydrochloride (0.0005%)
STERILE A
Hydrogen peroxide as preservative (0.004%)
Boric acid

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OPHTECS Europe B.V.